


Instructions


1




In order to avoid loosening of the crimped stent graft from the balloon, hold the protector at its orange marker with one hand and the delivery system with the other hand, then pull the protector off (see  8.4).


2



Ensure that the stent graft is not damaged (e.g. protruding struts) and that it is positioned between the RO-markers (don't touch the implant) (see  8.4).

3

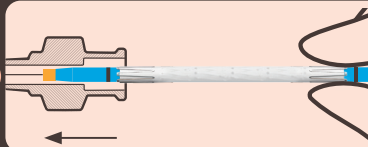

Carefully push the BeGraft Coronary Stent Graft System over the guide wire through the guiding catheter to reach the target lesion (see  8.6).

 Compliance Chart on the back

Compliance Chart

Inflation Pressure [bar]	Expanded Stent Outer Diameter [mm]						
	Ø 2.50	Ø 2.75	Ø 3.00	Ø 3.50	Ø 4.00	Ø 4.50	Ø 5.00
7						4.03	4.48
8	2.26	2.49	2.68	3.05	3.52	4.20	4.68
9	2.34	2.58	2.80	3.20	3.69	4.37	4.85
10	2.42	2.67	2.91	3.35	3.86	4.50	5.00
11	2.50	2.75	3.00	3.50	4.00	4.61	5.12
12	2.57	2.82	3.07	3.62	4.13	4.70	5.23
13	2.63	2.88	3.14	3.71	4.23	4.77	5.31
14	2.68	2.93	3.20	3.80	4.31	4.83	5.37
15	2.73	2.98	3.26	3.86	4.39		
16	2.78	3.02	3.32	3.92	4.46		

NP Nominal Pressure

RBP Rated Burst Pressure

Instructions on the back

