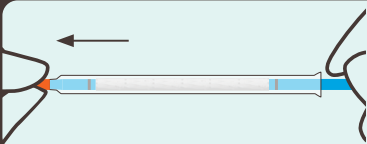

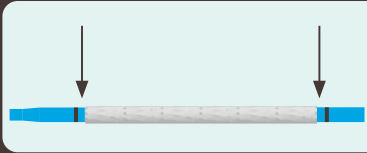

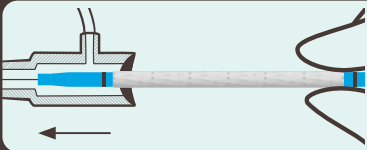



Instructions

- 

In order to avoid loosening of the crimped stent graft from the balloon, hold the protector at its orange marker with one hand and the delivery system with the other hand, then pull the protector off (see  7.3).
- 

Verify that the stent graft is located between the radiopaque balloon markers (see  7.3).
- 

Carefully hold the stent graft at the proximal end (with your fingers) when pushing it into the haemostatic valve (see  7.5).

 Compliance Chart on the back

Compliance Chart

Inflation Pressure [bar]	Expanded Stent Outer Diameter [mm]					
	Ø 5.0	Ø 6.0	Ø 7.0	Ø 8.0	Ø 9.0	Ø 10.0
8				8.0	9.0	10.0
9	5.0	6.0	7.0	8.3	9.2	10.2
10	5.2	6.2	7.2	8.5	9.4	10.4
11	5.3	6.4	7.3	8.7	9.6	10.6
12	5.4	6.5	7.5	8.8	9.7	10.7
13	5.5	6.6	7.6			

NP Nominal Pressure

RBP Rated Burst Pressure

Instructions on the back

