


Applicator Manual


1




Peel-off the flap to remove the protector from the Stent Graft (see  7.2).


2




Advance the yellow Applicator with the flared end to the back over the guide wire and position it in front of the introducer sheath (see  7.5).


3

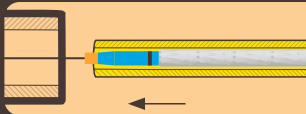

Advance the "BeGraft Aortic Stent Graft System" over the guide wire and place the Stent Graft completely inside the yellow Applicator (see  7.5).

4

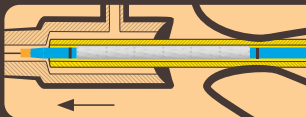

Push the yellow Applicator with the "BeGraft Aortic Stent Graft System" gently through the hemostatic valve of the introducer sheath. Further advance the "BeGraft Aortic Stent Graft System" into the introducer sheath. Retract the yellow Applicator to achieve hemostasis (see  7.5).

 Compliance Chart on the back

Compliance Chart

Inflation Pressure [bar]	Expanded Stent Outer Diameter [mm]						
	Ø 12	Ø 14	Ø 16	Ø 18	Ø 20	Ø 22	Ø 24
4	10.9	12.9	15.0	16.9	20.3	22.0	23.5
5	11.4	13.4	15.7	17.6	21.1	22.9	24.4
6	11.7	13.8	16.2	18.2	21.6	23.5	25.0
7	12.1	14.3	16.6	18.6			
8	12.3	14.5	16.9	18.8			
9	12.5	14.6	17.1				
10	12.6	14.8					

NP Nominal Pressure

RBP Rated Burst Pressure

Applicator Manual on the back

